



Actim[®] CRP

QUICK CRP RESULTS – FROM JUST ONE DROP

In just five minutes our **Actim CRP** rapid test helps estimate if an infection is viral or bacterial. You can get your results anywhere, simply and quickly – with no instrument needed. And all it takes is one small drop of blood.

Our test supports fast decisions on whether your patients require antibiotics – or more tests. And the simple sample is especially handy with children, who are often afraid of blood tests.

The Actim CRP rapid test is perfect for outpatient care, or anywhere you need reliable, on the spot results!

 Medix
Biochemica

 www.medixbiochemica.com

ACTIM CRP

QUICK RESULTS – ANYWHERE

Our Actim CRP test is extremely easy to use and kit has everything needed to perform the test. It delivers clear, accurate results in only five minutes. Test requires just 10 µl of whole blood from the fingertip. EDTA, citrate or heparin blood can also be used.

SPOT ON – AND ON THE SPOT

Actim CRP tests can be stored at room temperature, and they're packaged in single pouches – so there's no need to take the whole kit to your testing site. Likewise, there's no laboratory equipment or processing required – which means zero maintenance or calibrating.

HOW IT WORKS


Our rapid dipstick test is based on monoclonal antibodies highly specific to CRP. It can be used to help differentiate bacterial infections from viral infections.

Our test covers all clinically relevant CRP levels, with three different detection limits: ~10-40 mg/l, 40-80 mg/l and >80 mg/l. With these results, one can provide correct and fast treatment and avoid unnecessarily treating viral infections with antibiotics.



ABOUT CRP (C-reactive protein)

CRP is an acute-phase protein produced rapidly in response to inflammation, infection, and tissue damage. Level rises very quickly if an infection is present – rising higher in bacterial infections. CRP concentration can also fall very quickly if medical treatment is effective.

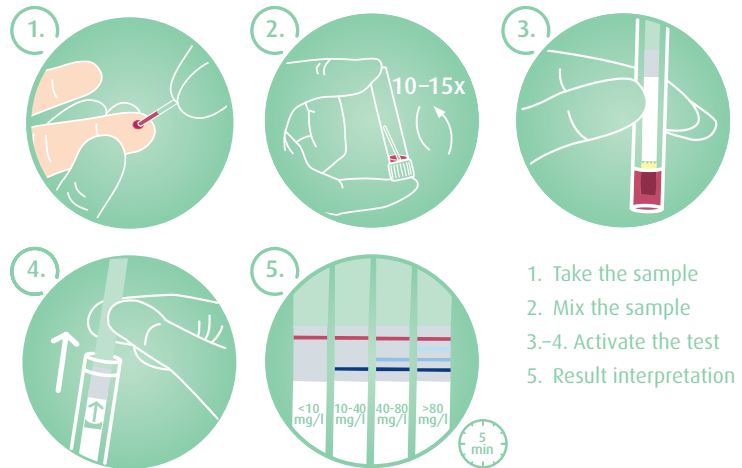

1. Take the sample
2. Mix the sample
- 3.-4. Activate the test
5. Result interpretation

CONTACT US

Medix Biochemica
Klovinpellontie 3, FI-02180 Espoo, Finland
medix@medixbiochemica.com

Ordering information

Actim CRP 20 test kit

31031ETAC

