Actim Combi[®] Hemoglobin. Transferrin.

FIND ALL GASTROINTESTINAL BLEEDING AT ONCE

Actim Combi Hemoglobin Transferrin is an easy-to-use rapid test for complete detection of gastrointestinal bleeding. The test detects two biomarkers for fecal occult blood: hemoglobin, which indicates bleeding in the lower gastrointestinal tract, and transferrin, a stable marker for occult bleeding in both the upper and lower digestive tract.

Thanks to the innovative sampling device, Actim Combi Hemoglobin Transferrin offers a hygienic and user-friendly testing solution for both patients and professionals. The simple and non-invasive procedure increases patients' willingness to take the test.

Actim Combi Hemoglobin Transferrin gives visually interpretable, accurate results in just 10 minutes. This enables early identification of patients who need further medical attention, while helping to avoid unnecessary and expensive diagnostic procedures.

HEMOGLOBIN:

The oxygen-carrying molecule in red blood cells; released into the digestive tract when gastrointestinal bleeding occurs.

TRANSFERRIN:

An iron-binding plasma glycoprotein. Like hemoglobin, transferrin enters stool through bleeding, but because it is resistant to proteolytic activity, it is less degraded in the gastrointestinal tract.

💥 www.medixbiochemica.com



Better coverage with dual detection

Dual detection of hemoglobin and transferrin reveals bleeding throughout the gastrointestinal tract.

No dietary restrictions

The test is based on antibodies specific against human hemoglobin and transferrin. Thus, no dietary restrictions are required before testing.

Clean and easy sample collection


The easy-to-use, leak-proof sample tube guarantees hygienic sample collection – even at home. The sampling device automatically adjusts the amount of sample. After sample collection, the tube can be stored at room temperature for up to one week.

Reliable and convenient testing

The innovative sample tube allows the whole test procedure to be carried out without opening the tube cap, which guarantees clean and user-friendly testing. Test results are consistent due to automatically adjusted amount of sample.

Immediate access to test results

With a test kit containing all necessary materials, and a one-step test protocol requiring only 15 seconds of hands-on time, Actim Combi test results are available in 10 minutes.

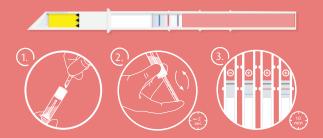

- 1. Perforation
- 2 Test activation
- 3 Result interpretation

CONTACT US

Oy Medix Biochemica Ab Tel. +358 9 547 680 www.medixbiochemica.com medix@medixbiochemica.com



Disclaimer: Product availability and/or labeling may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.