


Actim[®] Partus

THE RELIABLE
WAY TO IDENTIFY
AND RULE OUT
THE RISK OF
PRETERM DELIVERY


HOW TO USE ACTIM PARTUS

 Medix
Biochemica

 www.medixbiochemica.com


Actim Partus rapid test

- One-step dipstick test - very easy to use
- Can be used from 22 weeks onwards ^{4,6}
- Sampling takes only seconds
- Fast results at the bedside in just 5 minutes
- Intercourse, semen, urine, vaginal infections, vaginal medications, lubricants or bathing products do not interfere with test results ^{4,6,7}
- Test kit contains all the necessary materials

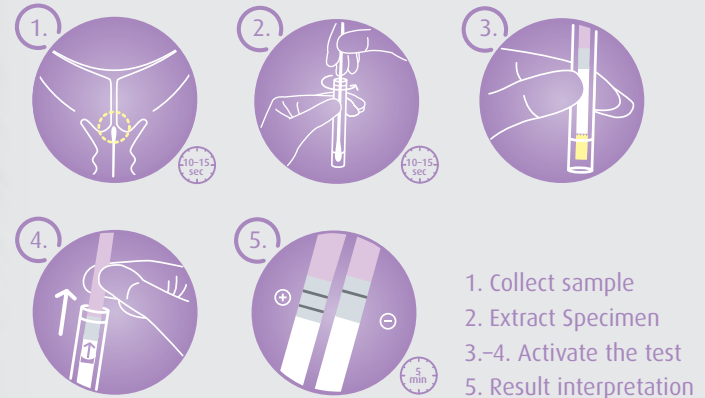

THE RELIABLE WAY TO IDENTIFY AND RULE OUT
THE RISK OF PRETERM DELIVERY

HOW TO USE ACTIM PARTUS

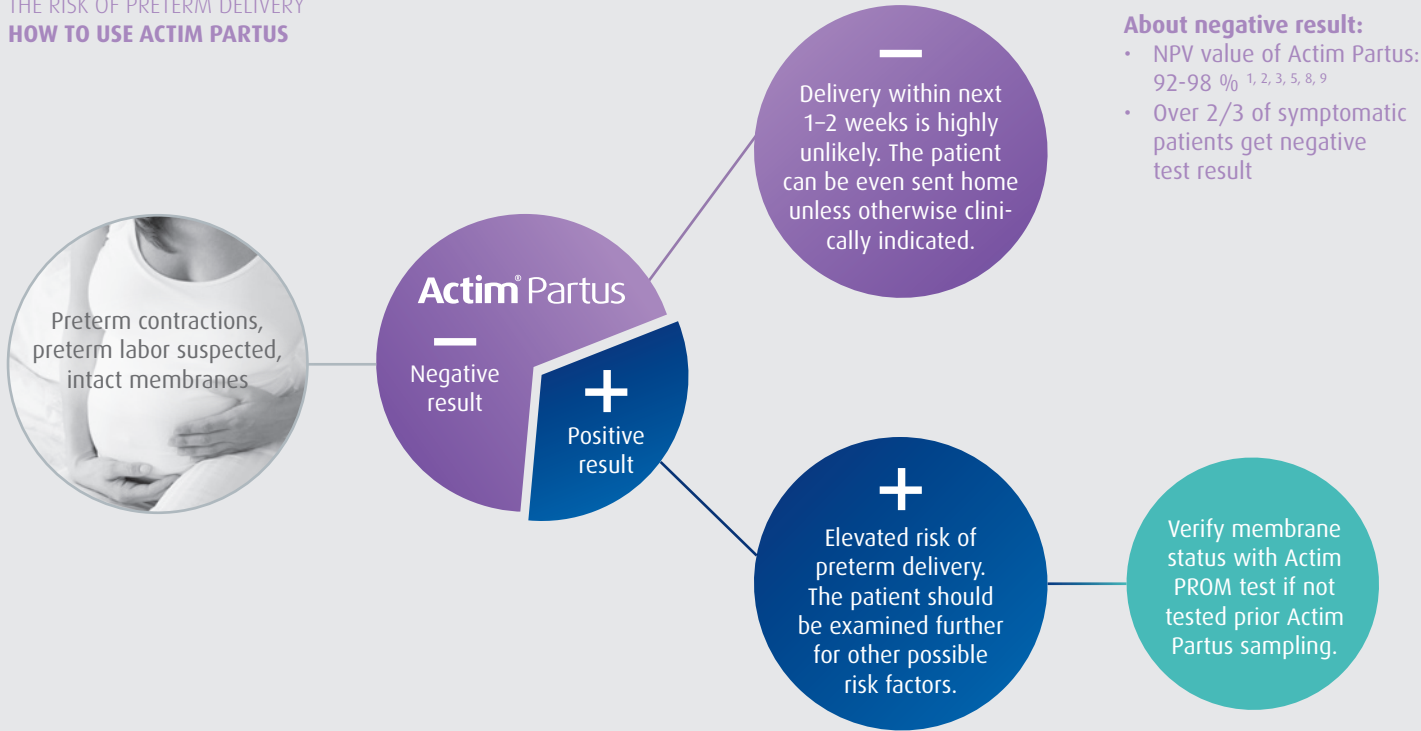

References

- 1) Azlin MI et al., Role of pIIGFBP-1 and ultrasound cervical length in predicting pre-term labour. Journal of Obstetrics and Gynaecology (2010) 30:456-460.
- 2) Brik Spinelli M et al., Phosphorylated insulin-like growth factor binding protein-1 and cervical measurement in women with threatening preterm birth. Acta Obstet Gynecol Scand (2010) 89:268-74.
- 3) Eroglu D et al., Prediction of preterm delivery among women with threatened preterm labor. Gynecol Obstet Invest (2007) 64:109-116.
- 4) Kekki M et al., Insulin-like growth factor-binding protein-1 in cervical secretion as a predictor of preterm delivery. Acta Obstet Gynecol Scand (2001) 80:546-551.
- 5) Lembet A et al., New rapid bed-side test to predict preterm delivery: phosphorylated insulin-like growth factor binding protein-1 in cervical secretions. Acta Obstet Gynecol Scand (2002) 81:706-712.
- 6) Rauhkonen L et al., Factors affecting decidual IGFBP-1 levels in the vagina and cervix in the first and mid-second trimester of pregnancy. BJOG (2009) 116:45-54.
- 7) Rutanen EM et al., Measurement of insulin-like growth factor binding protein-1 in cervical/vaginal secretions: comparison with the ROM-check Membrane immunoassay in the diagnosis of ruptured fetal membranes. Clinica Chimica Acta (1993) 214: 73-81.
- 8) Tanir HM, Sener T, Yildiz Z., Cervical phosphorylated insulin-like growth factor binding protein-1 for the prediction of preterm delivery in symptomatic cases with intact membranes. J Obstet Gynaecol Res (2009) 1:66-72.
- 9) Ting HS et al., Comparison of bedside test kits for prediction of preterm delivery: phosphorylated insulin-like growth factor binding protein-1 (pIGFBP-1) test and fetal fibronectin test. Ann Acad Med Singapore (2007) 36:399-402.