

Actim[®]
1ngeni

Taking point-of-care testing to
the next level with Actim[®] 1ngeni.



Actim 1ngeni is ingenious

Point-of-care testing becomes more convenient than before with Actim 1ngeni through:



Digital interpretation of the test results

With Actim 1ngeni, test results are automatically read at 5 minutes, allowing the healthcare staff to focus on the patient.

Furthermore, risks of human errors that might occur when test results are visually interpreted can be avoided with the instrument.



Improved traceability

Test results and key information such as patient IDs are automatically saved into the system for improved traceability at the point of care.



Data connectivity and operational efficiency

Test results can be conveniently transferred through LIS (Laboratory Information System) or AegisPOC™*.

*AegisPOC™ Point-of-Care Management Solutions is provided by Abbott.

Actim 1ngeni at a glance

Compatible
with Actim
PROM 1ngeni*

Compact design particularly convenient at the point of care or in the lab.

Test results can be saved to a USB memory stick or printed with a USB printer if needed.

Capacity to store up to 999 test results.

Intuitive touchscreen interface available in 15 languages.

Relevant data such as user ID, patient ID, test lot, date & time of the test procedure can be saved into the instrument.

A barcode reader or a touch screen can be used to input data.

Up to 99 authorized users.



Actim 1ngeni reads the QR code on the cartridge that contains the test and lot-specific data.

Quality Controls Options Available for Actim 1ngeni




The functionality of Actim 1ngeni can be monitored via instrument self-test and instrument check.




The functionality of the test can be monitored with controls that are specific to the test in use.

*Actim PROM 1ngeni, 10 tests | product code: 30831RETAC.

How to use Actim 1ngeni in 8 easy steps*

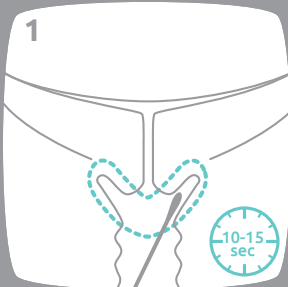



1. Collect the specimen with or without a speculum.

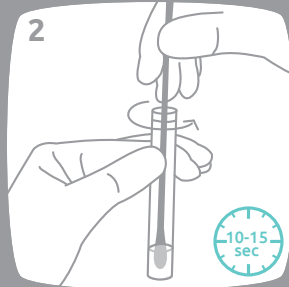



2. Extract the specimen




3. & 4. Prepare the instrument.

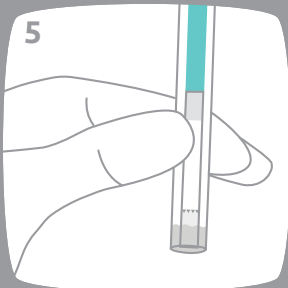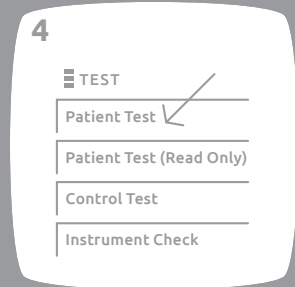

5. & 6. Activate the test

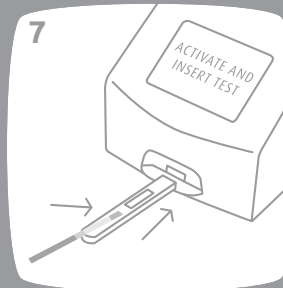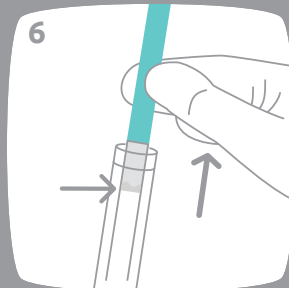

7. Start the procedure

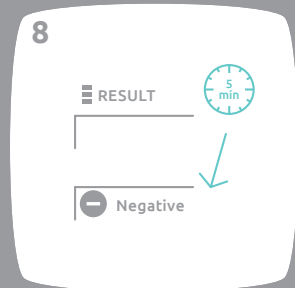

8. Read the test result from the screen

*Test procedure with Actim PROM 1ngeni.

Ordering information

Actim Oy

– part of Medix Biochemica Group
Klovinpellontie 3, FI-02180 Espoo, Finland
actim@actimtest.com

- ✓ Actim 1ngeni instrument | product code 19101AC
 - ✓ Actim 1ngeni Instrument Check I product code 19116IC
- Test and Control kits are available separately:**
- ✓ Actim PROM 1ngeni, 10 tests | product code 30831RETAC
 - ✓ Actim PROM Controls I product code 30800ETAC

www.actimtest.com